MAGNESIUM BR

CALMAGITE


Colorimetric method

ENDPOINT

<p>REF CT10312 3 x 80 mL</p> <p>CONTENTS R1.Reagent 3 x 40 mL R2.Reagent 3 x 40 mL</p>
For <i>in vitro</i> diagnostic use only

PRINCIPLE

The method¹ is based on the specific binding of calmagite, a metallochromic indicator², and magnesium at alkaline pH with the resulting shift in the absorption wavelength of the complex. The intensity of the chromophore formed is proportional to the concentration of magnesium in the sample.

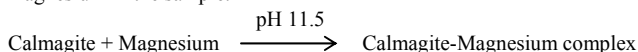REAGENTS

R1. Chromogen. Calmagite 70 mg/mol/L, EGTA 50 mmol/L, sodium chloride 800 mmol/L, surfactant 0.05 % (w/v).

R2. Alkaline buffer. Dietanolamine 1 mg/mol/L, AMP 100 mmol/L, potassium cyanide 10 mmol/L, non-reactive stabilizers.

Xi R:36 S :22-26

REAGENTS PREPARATION

The reagents are ready to use.

STORAGE AND STABILITY

Store at 2-8°C. The reagents are stable until the expiry date stated on the label. On board the reagents are stable 30 days. After daily use store tightly closed and protected from light.

Discard if appear signs of deterioration:

- Presence of particles and turbidity.
- Blank absorbance (A) at 520 nm > 1.000 against distilled water.

SAMPLE COLLECTION

Serum or heparinized plasma free of hemolysis. Other anticoagulants (EDTA, oxalate and citrate) must not be used. Magnesium in serum or plasma is stable for 10 days at 2-8°C. Freeze for longer storage.

INTERFERENCES

- Lipemia (intralipid) may affect the results.
- Bilirubin (40 mg/dL) does not interfere.
- Hemoglobin (> 2 g/L) may affect the results.
- Other drugs and substances may interfere³.
- Many detergents and water supplies (see Notes).
- The interference by calcium is prevented by the use of EGTA into the buffer.

INSTRUMENTATION AND MATERIALS

- LIDA analyzer.
- Laboratory equipment.
- iso-Clean Solution. Ref. CT18002.
- Multicalibrator CC/H 5x5 mL. Ref. CT19750

AUTOMATED PROCEDURE

A graphic display pictures the specific sets corresponding to the technical application outlined for this test.

Any new application, to the instrument should be validated to confirm that results meet the analytical performance of the method.

It is recommended to validate periodically the instrument.

CALIBRATION

Calibration stability on board is kept for 8 days. An abnormal increase of calibration resets is usually indicative of failure in the performance of the reagents. Recalibrate when a new lot of reagents are used, when control recovery falls out range and after any adjustment made to the analyzer. A reagent blank should be run daily before sample analysis.

RESULTS

Samples with concentrations higher than 10 mg/dL should be diluted 1:2 with saline and assayed again. Multiply the results by 2.

If results are to be expressed as SI units apply: mg/dL x 0.412 = mmol/L

EXPECTED VALUES⁴

Serum, plasma

Children (2-12 years)	1.7-2.3 mg/dL (0.70-0.94 mmol/L)
Adults (12-60 years)	1.6-3.0 mg/dL (0.66-1.23 mmol/L)

It is recommended that each laboratory establishes its own reference range.

QUALITY CONTROL

To ensure adequate quality control (QC), each run should include a set of controls (normal and abnormal) with assayed values handled as unknowns. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

DIAGNOSTIC CHARACTERISTICS

Magnesium is considered an essential nutrient and a major intracellular cation. More than 50% of the total magnesium found in the body is complexed with calcium in the skeleton; however, only 1% of total body magnesium is found in the circulation. Between 60% and 70% of serum magnesium is free; the remaining percentage is bound to albumin, phosphate, citrate, and other ions. *Hypomagnesemia* (an abnormal decrease in the level of serum magnesium) is usually associated with severe prolonged diarrhea, impairment of neuromuscular function, gastrointestinal malabsorption and alcoholism. *Hypermagnesemia* (an abnormal elevation in the level of serum magnesium) is usually associated with renal glomerular failure, dehydration, severe diabetic acidosis, and Addison's disease. Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

NOTES

- Most of the detergents and water softening products used in the labs contain chelating agents. A defective rinsing will invalidate the procedure. Keep the glassware acid washed and thoroughly rinsed at all times.

PERFORMANCE CHARACTERISTICS

Performance characteristics are available on request.

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A Standard

REAGENT PREPARATION

Working reagent Mix 1 volume of R1 + 1 volume of R2 Stable for 1 week at room temperature (16-25°C). Protect from light.

MANUAL PROCEDURE

- Bring reagents and samples to room temperature.
- Pipette into labelled test tubes:

TUBES	Blank	Sample	Standard
Working reagent	1.0 mL	1.0 mL	1.0 mL
Sample	-	10 µL	-
Standard	-	-	10 µL

- Mix and let the tubes stand 2 minutes at room temperature.
- Read the absorbance (A) of the samples and the standard at 520 nm against the reagent blank.

The color is stable for at least 1 hour.

CALCULATIONS

Serum, plasma

$$\frac{A_{\text{Sample}}}{A_{\text{Standard}}} \times C_{\text{Standard}} = \text{mg/dL magnesium}$$

A Standard